An SDMA case study: Scarlet





Patient: Scarlet, 5-year-old, spayed female golden retriever

Presenting reason: Scarlet was brought in for her annual checkup and vaccines.

History: Scarlet's owners shared that she had not been acting like herself the past few weeks.

She had a couple of urinary tract infections in the past, though she had not recently been showing those types of signs. She had been less energetic, but appetite and drinking had seemed normal as far as they had been able to tell.

Physical examination: Scarlet was bright, alert, and responsive (BAR), appeared well-hydrated, and had good body condition with stable weight since last visit.

Diagnostic plan

Complete blood count (CBC); chemistry panel, including the IDEXX SDMA $^{\rm TM}$ Test and electrolytes; complete urinalysis; and total T₄ were recommended.

Laboratory findings

- Scarlet's CBC was unremarkable, though her other blood and urine diagnostics identified some areas of concern.
- She had an **increased SDMA* of 17 µg/dL (0–14 µg/dL)** and a **low urine specific gravity of 1.026.** Her creatinine was within the reference interval at 1.5 mg/dL.
- Her total T₄ and free T₄ were both within reference intervals but were low normal.

Treatment plan and follow-up

- Scarlet was started on L-thyroxine for treatment of presumptive hypothyroidism.
- A recheck was scheduled in 2–3 weeks for follow-up CBC, chemistry, total T₄, and complete urinalysis.
- Additional test considerations included diagnostic imaging of her kidneys and associated structures as well as evaluation of urine protein:creatinine (UPC) ratio and blood pressure.

2-week recheck: The pet owner reported that Scarlet seemed to be more active. Her total T₄ result was well within the normal range, her SDMA remained increased at 18 μ g/dL, and her creatinine remained within the reference interval. Her urine specific gravity remained low at 1.027.

Chemistry, December 31

+		31/2014 (Order Received) /2015 @ 5:49 PM (Last Updated)	IDEXX Reference Laboratories
▶ Glucose	91	63 - 114 mg/dL	
▶ BUN	29	9 - 31 mg/dL	
Creatinine	1.5	0.5 - 1.5 mg/dL	
IDEXX SDMA Learn More	^m 17	0 - 14 µg/dL	
BUN:Creatinine Ratio	19.3		
Phosphorus	3.0	2.5 - 6.1 mg/dL	
Calcium	10.3	8.8 - 11.2 mg/dL	
▶ Sodium	145	142 - 152 mmol/L	
Potassium	4.4	4.0 - 5.4 mmol/L	
Na:K Ratio	33	28 - 37	
Chloride	111	108 - 119 mmol/L	
TCO2 (Bicarbonate)	20	13 - 27 mmol/L	
Total Protein	6.7	5.5 - 7.5 g/dL	
▶ Albumin	3.4	2.7 - 3.9 g/dL	
▶ Globulin	3.3	2.4 - 4.0 g/dL	
> Cholesterol	473	131 - 345 mg/dL	

Urinalysis, December 31

	+	12/31/2014 (Order Received) 1/6/2015 @ 5:49 PM (Last Updated)	IDEXX Reference Laboratories	
Collection		CYSTOCENTESIS		
▶ Color		YELLOW		
Clarity		CLEAR		
Specific Gravity		1.026		
⊳ pH		7.0		
Protein		NEGATIVE		
Glucose		NEGATIVE		
Ketones		NEGATIVE		
Blood / Hemoglobin		NEGATIVE		
Bilirubin		NEGATIVE		
Urobilinogen		NORMAL		
White Blood Cells		NONE SEEN		
Red Blood Cells		2-5		
Bacteria		NONE SEEN		
Epithelial Cells		1+ (1-2)		
Mucus		NONE SEEN		

Total T4 and Free T4, December 31

	+		12/31/2014 (Order Received) 1/6/2015 @ 5:49 PM (Last Updated)	IDEXX Reference Laboratories Show Details
Total T4		^d 1.1	1 - 4 µg/dL	
Free T4 (ng/dL)		0.6	0.6 - 3.7 ng/dL	
Free T4 (pmol/L)		^e 7.7	7.7 - 47.6 pmol/L	

Action plan and results

- Because of the **persistent SDMA elevation**, an ultrasound on Scarlet's kidneys and associated structures was scheduled.
- The ultrasound showed the left kidney was somewhat rounded and mildly irregular with some mild dilation of the renal pelvis with changes suspected to be the result of prior kidney infections.
- A urine culture was negative; the UPC was 0.0 and her average systolic blood pressure was normal at 140 mm Hg.
- Scarlet continued to show a persistence in SDMA elevation and abnormal kidney imaging. Following the International Renal Interest Society (IRIS) Chronic Kidney Disease (CKD) Staging Guidelines, these findings supported IRIS CKD Stage 2 disease, which was substaged as normotensive and nonproteinuric.
- Scarlet's treatment plan included initiating a kidney therapeutic diet and continuing with her L-thyroxine and access to fresh water sources at home, along with more regular appointments to monitor her kidney health. A 3-month recheck was scheduled to perform CBC, chemistry panel, total T₄, and complete urinalysis.

3-month recheck: Scarlet was reportedly doing well at home. Her total T_4 levels remained well within the reference interval, and CBC was unremarkable. Her SDMA was 15 μ g/dL, and her creatinine remained within the normal range. Her urine specific gravity remained low at 1.027.

6-month recheck: Given Scarlet's activity level, improving diagnostics, and reports from the pet owner, twice yearly checkups were recommended moving forward. At that time, **Scarlet's SDMA remained at 15 μg/dL**, her creatinine was still within the normal range, and her urine specific gravity was at 1.026.

Discussion

SDMA is a more reliable biomarker for kidney disease than creatinine, and it helped to define a thorough medical follow-up plan for Scarlet, even at early stages of kidney changes.

Early intervention and support of Scarlet's kidney function with a kidney supportive diet helped lead to stabilized SDMA results in her follow-up rechecks. Evidence shows that SDMA can be relied upon for early detection of kidney disease and that dietary intervention prior to an elevation of creatinine can have health benefits for the kidneys.^{1,2}

Chemistry, January 18

		/2015 (Order Received) /2015 @ 4:08 PM (Last Updated)	IDEXX Reference Laboratories Show Details
▶ Glucose	92	63 - 114 mg/dL	
▶ BUN	28	9 - 31 mg/dL	
▶ Creatinine	1.6	0.5 - 1.5 mg/dL	
► IDEXX SDMA ► Learn More	18	0 - 14 µg/dL	
BUN:Creatinine Ratio	17.5		
Phosphorus	3.8	2.5 - 6.1 mg/dL	
Calcium	10.8	8.8 - 11.2 mg/dL	
▶ Sodium	149	142 - 152 mmol/L	
Potassium	4.6	4.0 - 5.4 mmol/L	
Na:K Ratio	32	28 - 37	
▶ Chloride	112	108 - 119 mmol/L	
TCO2 (Bicarbonate)	22	13 - 27 mmol/L	
Anion Gap	20	11 - 26 mmol/L	

Total T4, January 18

	+	1/18/2015 (Order Received) 1/20/2015 @ 4:08 PM (Last Updated)		IDEXX Reference Laboratories Show Details	
Total T4		° 2.3	1 - 4 µg/dL		

Chemistry, March 27

		7/2015 (Order Received) L/2015 @ 5:59 PM (Last Updated)	IDEXX Reference Laboratories Show Details
Glucose	89	63 - 114 mg/dL	
BUN	24	9 - 31 mg/dL	
Creatinine	1.5	0.5 - 1.5 mg/dL	
IDEXX SDMA Learn More	^g 15	0 - 14 µg/dL	
BUN:Creatinine Ratio	16.0		
Phosphorus	3.9	2.5 - 6.1 mg/dL	
Calcium	10.3	8.8 - 11.2 mg/dL	
Sodium	148	142 - 152 mmol/L	
Potassium	4.8	4.0 - 5.4 mmol/L	
Na:K Ratio	31	28 - 37	
Chloride	113	108 - 119 mmol/L	
TCO2 (Bicarbonate)	20	13 - 27 mmol/L	
Anion Gap	20	11 - 26 mmol/L	

*Symmetric dimethylarginine

References

 Hall JA, MacLeay J, Yerramilli M, et al. Positive impact of nutritional interventions on serum symmetric dimethylarginine and creatinine concentrations in client-owned geriatric dogs. PLoS One. 2016;11(4):e0153653

 International Renal Interest Society. IRIS Staging of Chronic Kidney Disease (CKD) Guidelines www.iris-kidney.com/pdl/staging-of-ckd.pdf. Modified 2015. Accessed June 23, 2016.



